Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKU PRO INSTALACI

Sagitta kids Sagitta SI kids

Contents

Preface	4
1. This product	5
2. Scope of delivery	6
3. Assembly and adjustments	6
3.1. Tools	7
3.2. Adjusting the wheelchair to the user	7
3.3. Changing pneumatic tyres	16

Preface

To support you on the installation and repairs of this wheelchair, this manual is offered. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following wheelchair(s):

- Sagitta kids
- Sagitta SI kids

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions (Instructions for detail adjustments) For specialist dealer
	Torque table (maximum torques for bolts and nuts) For specialist dealer
	Service manual for wheelchairs For specialist dealer
	Drawings of (spare) parts For specialist dealer

1. This product

EN

1. Handgrips
2. Backrest
3. Side plate / Armrest
4. Seat
5. Footrests
6. Footplates
7. Front wheels
8. Brakes
9. Rear wheels
10. Hand rims
11. Identification plate

2. Scope of delivery

The following items are part of the delivery.

- 1 frame with splash guard, rear and front wheels, foot support
- Tools
- Manual
- Accessories

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

3. Assembly and adjustments

CAUTION

Risk of injuries or damage

- The wheelchair needs to be adjusted by a specialist dealer according to the instructions in this manual.
- Any modification to the product will lapse the warranty and will cease the responsibility of Vermeiren.
- Only use parts and tools described in this manual.
- Prevent bystanders from accessing the working area.
- While adjusting and operating the wheelchair, make sure no objects or body parts get caught between moving parts.
- Only use the settings described in this manual.
- Make sure that the adjustment on left and right side of the wheelchair are the same.
- Deviation of the adjustment settings may influence the stability of the wheelchair with risk of tilting and falling.
- Make sure all screws are firmly secured before riding the wheelchair. Adhere to the tightening torques for screws according to the Torque table on our website.

-  Read the user manual of this wheelchair. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

3.1. Tools

To (dis)assemble and adjust the wheelchair, the following tools are needed:

- Wrench no. 10, 11
- Allen key no. 4, 5

3.2. Adjusting the wheelchair to the user

3.2.1. Seat height


WARNING

Risk of tipping over


- Make sure that both steering wheels are mounted with an identical front fork.
- Make sure that both steering wheels are mounted in the same position in the front fork.

The Sagitta kids wheelchair is adjustable in different seat heights by changing the position of the front wheels, diameter of the front wheels, and diameter of the rear wheels.


Seat height without cushion	Rear wheels	Front wheels		
	Ø	Ø	Fork	Hole
370 mm	18"	3"	Short	1
375 mm	20"	3"	Medium	1
400 mm	20"	4"	Medium	2
403 mm	22"	3"	Long	1
428 mm	22"	4"	Long	2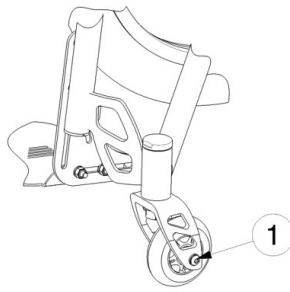


Seat height 370 mm, short front fork

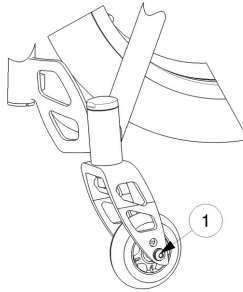
Seat height 375 mm, medium front fork

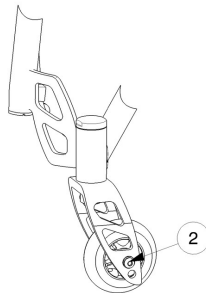Seat height 400 mm, medium front fork

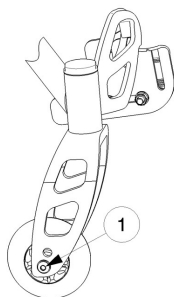Seat height 403 mm, long front fork

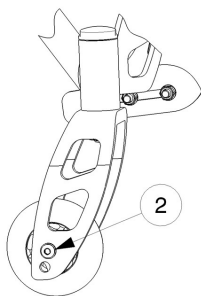Seat height 428 mm, long front fork

To change the seat height:

1. Remove the rear wheels (see instruction manual).
2. Exchange the rear wheels for the necessary diameter.
3. Insert the rear wheels.
4. Unscrew the swivel axles of the front wheels.
5. Choose the correct diameter for the front wheels, see table.
6. Place the front wheels in the right hole of the front fork, see table.
7. Install the swivel axles of the front wheels and tighten. Check the tension in the swivel axle so it turns smoothly, but there is no space for motion.

If installed properly, the swivel axles are perpendicular to the ground.

8. Adjust the brakes accordingly, see §3.2.5..

3.2.2. Seat width


⚠ WARNING

Risk of injuries or damage

- Make sure that both sides of the frame are positioned in the same hole.

Seat width	Position of hole (Fig. C)
240 mm	1
260 mm	2
280 mm	3
300 mm	4
320 mm	5
340 mm	6

Fig A

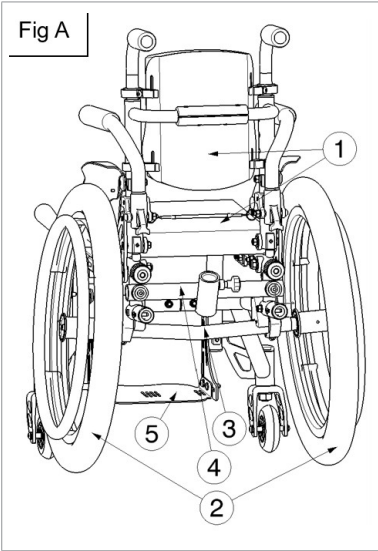A technical line drawing of a wheelchair frame from a rear three-quarter perspective. Numbered callouts point to specific components: 1 points to the backrest cushion, 2 points to the rear wheels, 3 points to the rear axle, 4 points to the push bar assembly, and 5 points to the footplate. Arrows indicate the removal of these parts.

1. Remove the seat and back cushions (1).
2. Remove the rear wheels (2).
3. Remove the axle of the rear wheels (3).
4. Remove the axle to attach the push bar (if available) (4).
5. Remove the footplate (5).
6. According to the desired seat width, use the correct dimensions of the cushions, rear wheel axle, push bar axle, and footplate. (When changing the seat width, you need to buy the correct parts.)

Fig B


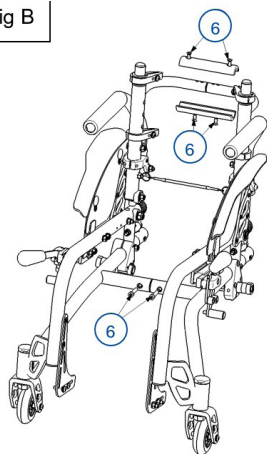
Fig C

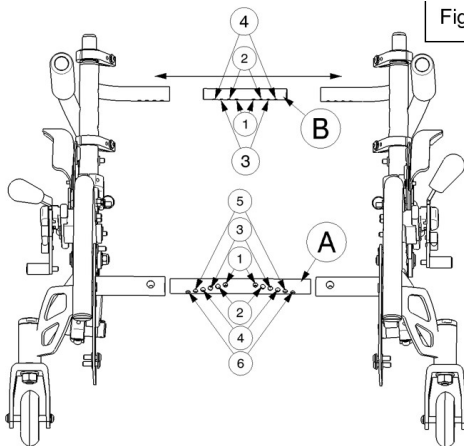
7. Remove the screws (6). (Fig. B)
8. Adjust the frame to the desired seat width by placing the left and right side frames in the correct hole of tube (A) and (B). For seat widths 320mm and 340mm, you need to use another push bar tube (B). (Fig. C)
9. Retighten the screws (6) properly by hand. (Fig. B)
10. Mount all removed parts (cushions, axles, rear wheels, footplate) with the correct dimensions. (Fig. A)

3.2.3. Seat angle (Sagitta *SI kids*)

The Sagitta *SI kids* wheelchair can be adjusted in 6 different seat angles without changing wheel diameters and front wheel forks.


Seat depth	Seat angle	Hole
280 - 300 - 320 mm	4°	1
280 - 300 - 320 mm	5°	2
280 - 300 - 320 mm	6°	3
280 - 300 - 320 mm	7°	4
280 - 300 - 320 mm	8°	5
280 mm	10°	6
300 - 320 mm	9°	6
280 - 300 - 320 mm	Not used	7

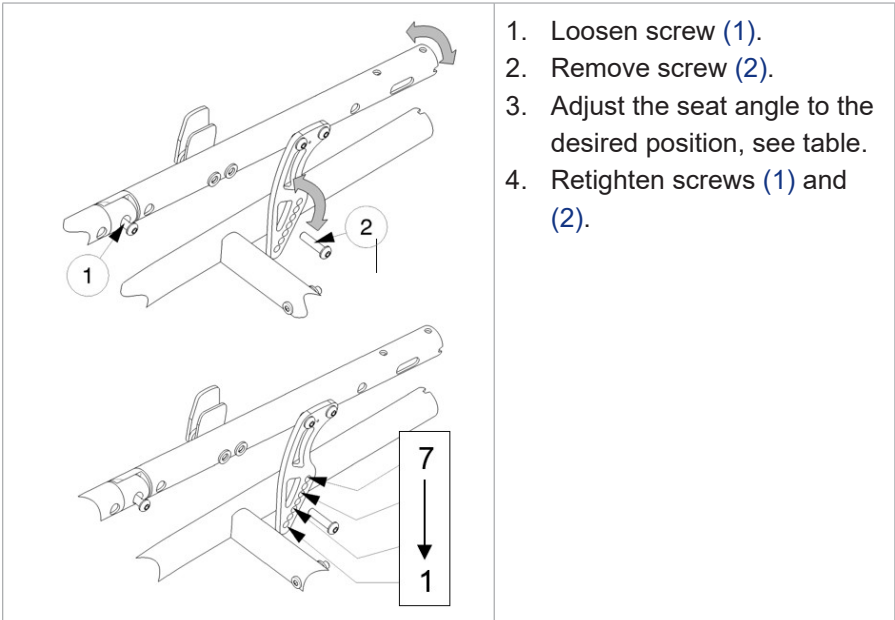3.2.4. Backrest height

The backrest height can be adjusted in 6 different positions: two by changing the frame, and three by changing the position of the back tube.

Frame	Changing position of the back tube		
	Hole 1	Hole 2	Hole 3
1	275 mm	300 mm	325 mm
2	325 mm	350 mm	375 mm

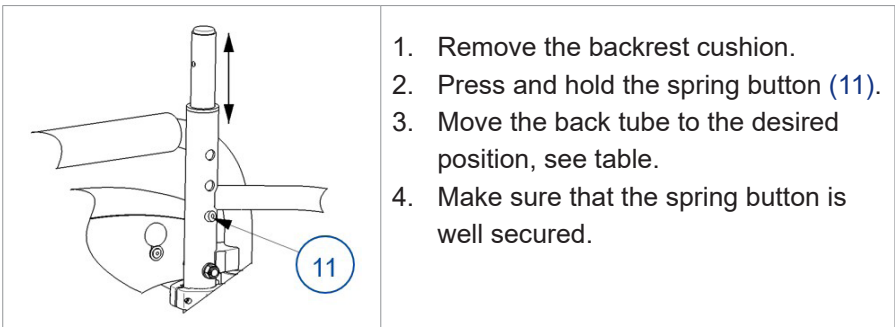
3.2.5. Adjusting the brakes

⚠ CAUTION

Risk of injuries

- Always adjust the parking brakes after adjusting the rear wheel position. Readjust also when tyres are worn, to compensate the wear.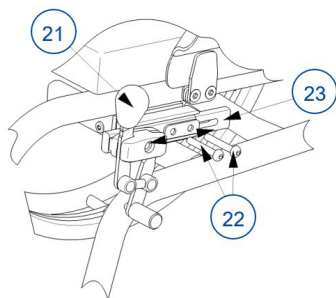


1. Release the brake with lever (21), see instruction manual.
2. Loosen the bolts (22) so the brake mechanism can slide over the guide (23). Pull the brake mechanism to the desired position.
3. Retighten the bolts.
4. Verify if the brake is working properly: the wheel is blocked, but the brake can be operated easily. If this is not the case, repeat the steps above.
5. Repeat this adjustment for the second parking brake. Make sure both brakes are adjusted identically.

3.2.6. Footrest length

⚠ CAUTION


Risk of injuries or damage

- Make sure that the footrests won't touch the ground. Keep a minimum safe distance of 60 mm above the ground.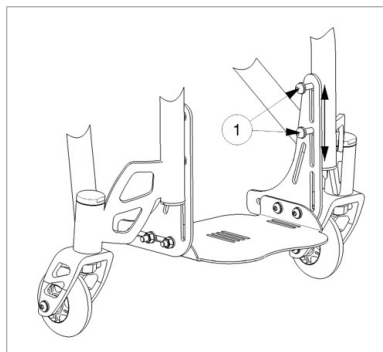

1. Loosen screws (1) on both sides of the footrest.
2. Adjust the footrest to a comfortable length.
3. Retighten the screw firmly.


3.2.7. Footrest depth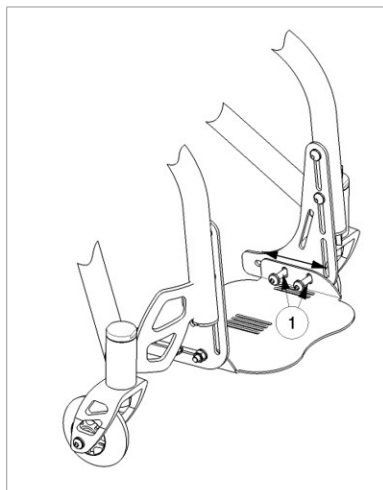

1. Remove the screws (1).
2. Adjust the footrest depth to a comfortable position.
3. Retighten the screw firmly.

3.2.8. Rear wheel camber

The angle of the rear wheels is adjustable (0° , 2° , 4° , 6° , 8° , 10°) by changing the axle (1) of the rear wheels.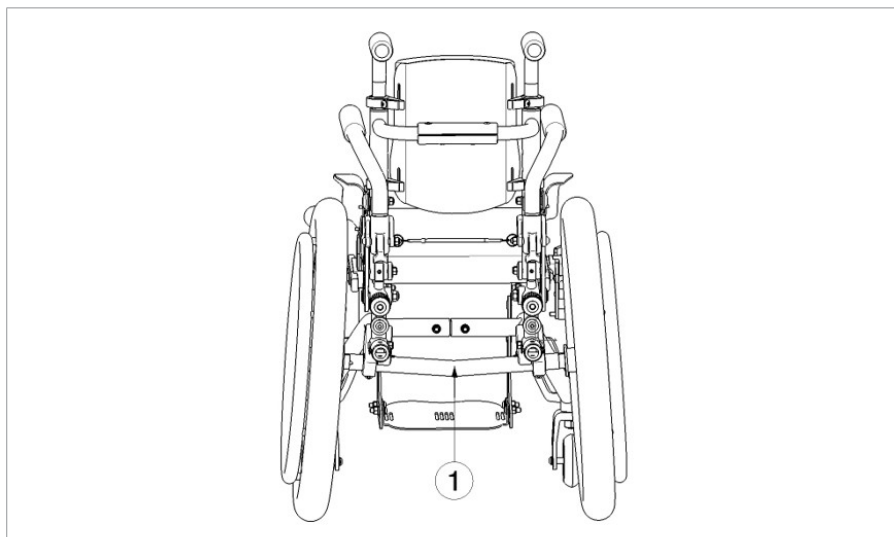


3.2.9. Stability and manoeuvrability

⚠ WARNING

Risk of injuries

- Be aware that these adjustments have an influence on the stability of the wheelchair.
- Make sure that there can be no collision with the steering wheels.
- Make sure to adjust the brake position accordingly, see §3.2.5..

The stability and manoeuvrability can be adjusted by moving the axle block of the rear wheels (2) more to the front or the back of the frame.

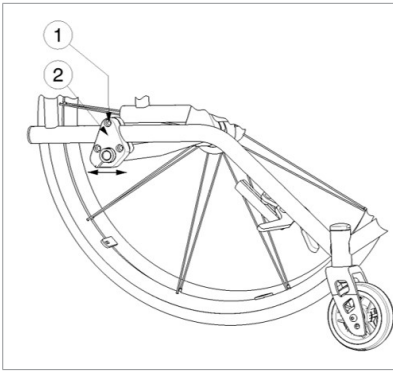1. Loosen the upper screw (1) of the axle block.
2. Move the axle block (2) to the desired position.
3. Retighten the screw firmly.

3.3. Changing pneumatic tyres

⚠ CAUTION

Risk of injuries or damage

- Make sure that all air is released from the tyre, before removing it.
- Make sure that no body parts, the inner tube nor any other objects are pinched between tyre and rim.
- Make sure all screws are firmly secured before driving.
- Make sure that the pressure is correct, see the pressure indication on the tyres.
- Inflate the tyre using inflating equipment which complies to regulations and with a pressure indication in “bar”.
- Only use genuine spare or replacement parts approved by Vermeiren.

To change the tyre or inner tube, proceed as follows:

1. Let all the air out of the inner tube.
2. Insert a tyre lever between the tyre and the rim.
3. Gently push the tyre lever downwards. This will pull the tyre over the edge of the rim.
4. Move the lever along the rim so the tyre pops out.
5. Carefully remove the tyre from the rim and remove the tube.

	Place the rim belt in position over the air valve before inserting it into the rim. The rim belt can now be pulled over easily. Check that all spoke heads are covered (a rim belt is not required in the case of a plastic rim).
	Push the tyre over the edge of the rim, starting behind the air valve. Inflate the tube slightly until it is round, and place it inside the tyre.
	If the tube fits inside the tyre without any folds (in case of folds: let out some air), then the upper side of the tyre can be pressed lightly onto the rim with both hands, starting at the air valve. Lightly push the air valve inwards and pull it out again to make sure that the tyre is positioned properly in the region of the air valve.

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All rights reserved, including translation.