

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

### B15A TABLE FOR ELECTROS

1/2

Scale:	1:5	Drafted	Verified	Confirmed
Date	24.06.2024	09/07/2024	09/07/2024	09/07/2024
Initials	AZ	PM	MK	


---

Plan No.  
E00083909/B.1

\* - The B15A table should be mounted on the opposite side from the controller.


No.	Plan	Article	Qty
1	Y002759	1217101	1
2	Y011782	1678501	1
3	YP04677	2887154	1
4	X000989	1004539	4
5	X001101	1001605	2
6	XP00149	2990111	1
7	M6 DIN125A	1000973	8
8	RIVET 5x18	1001589	2
9	M6x20 ISO7380	1006999	4
10	RIVET 4x14	1001584	4
11	Rubber 2mm	1003299	2,10m
12	P000025	---	1

Point of rubber corner connection

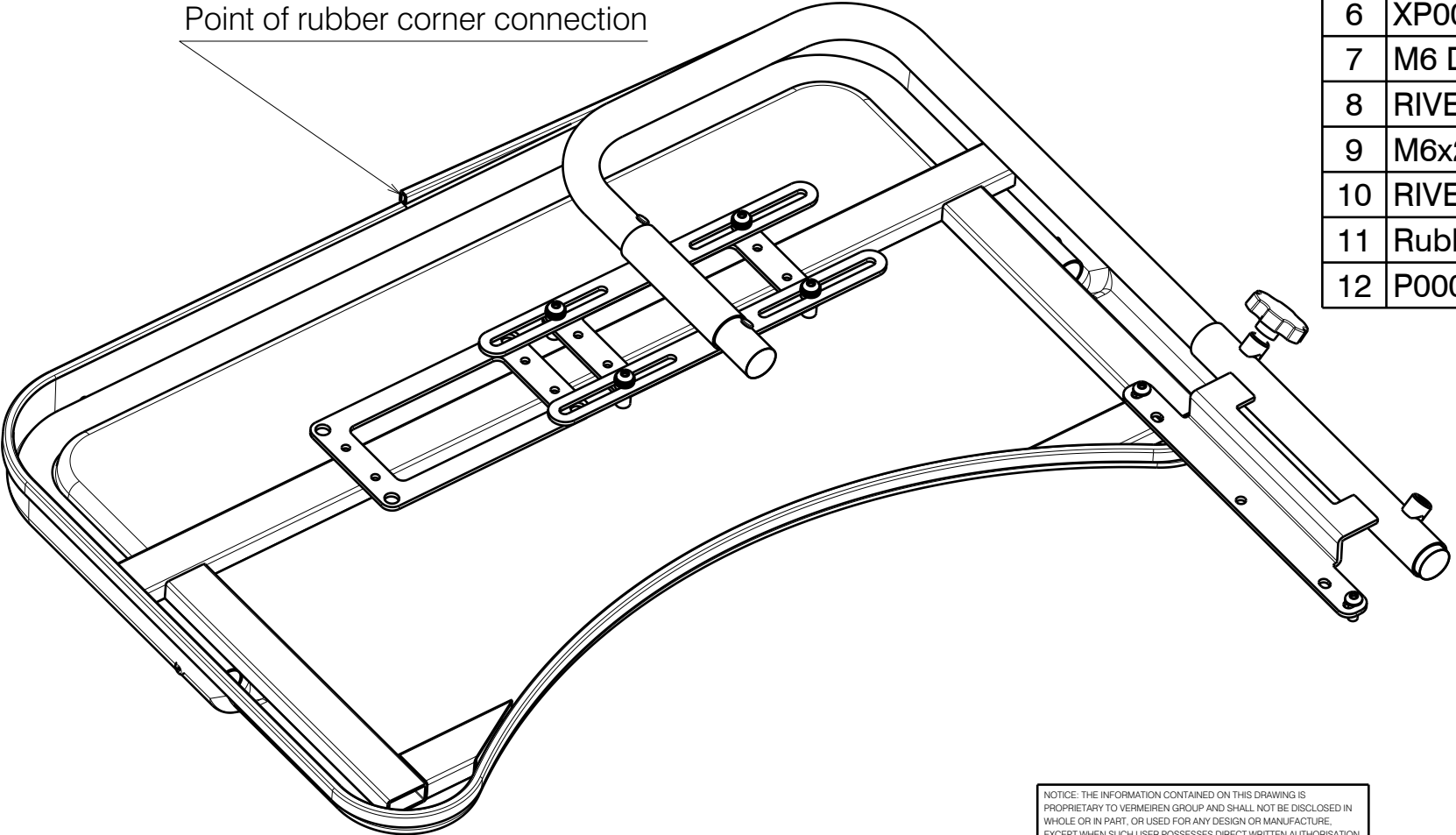

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

B15A TABLE FOR ELECTROS				2/2
Scale:	1:5	Drafted	Verified	Confirmed
Date	24.06.2024	09/07/2024	09/07/2024	
Initials	AZ	PM	MK	

---
Plan No. E00083909/B.1


\* - The B15A table should be mounted on the opposite side from the controller.