


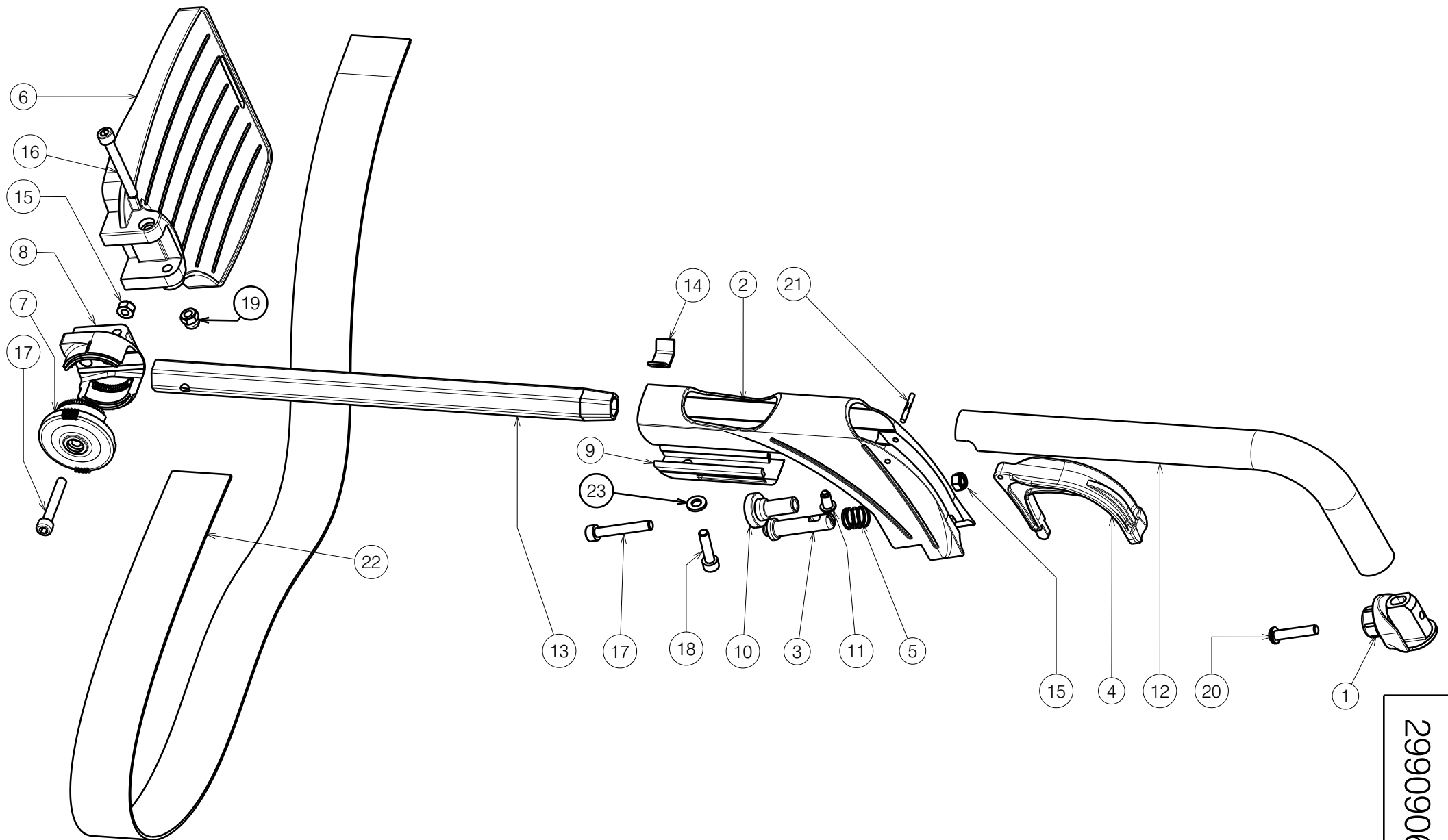

Sheet 1/2

2990906

E000629 / M

ADJUSTABLE FOOTREST B06 V200/V300/V300DL				Plan No.
				E000629 / M
Scale: 3:10	Drafted	Verified	Confirmed	
Date	28/04/2010	16/12/2024	16/12/2024	
Init.	TM	PM	AS	

Material			
No.	Plan	Article	Quantity
1	X004414	1904130	1
2	X004416	1904202	1
3	X004417	1719200	1
4	X004418	1904207	1
5	X004423	1717300	1
6	X004570	1904646 (L)	(39-42) 1
	X004571	1904647 (R)	(39-42) 1
	X004457	1904475 (L)	(44-46) 1
	X004453	1904210 (R)	(44-46) 1
	X004572	1904648 (L)	(48-60) 1
	X004573	1904649 (R)	(48-60) 1
6*	XC00158	3991003 (L)	(39) 1
	XC00157	3991002 (R)	(39) 1
7	X004464	1904212	1

* Only for Express

Article numbers for sets			
No.	Plan	Article	Quantity
I	Butt set	17416	2
II	Footplate 39-42	1906153	1
	Footplate 44-46	1906154	1
	Footplate 48-50	1906155	1

Material			
No.	Plan	Article	Quantity
8	X004465	1904213	1
9	X004658	1904968	1
10	X004660	1905195	1
11	X004687	1905639	1
12	Y015007	17300..	1
13	Y016499	F006269	1
14	Y016501	1733500	1
15	M6 DIN985	1001680	2
16	M6x50 DIN912	1007970	1
17	M6x40 DIN912	1003909	2
18	M6x25 DIN912	1001494	1
19	Nut M6	1008492	1
20	M6x30 ISO7380	1009588	1
21	D3x24 DIN1475	1905640	1
22	---	1905642	1
23	M6 DIN125	1001564	1

Sheet 2/2

2990906

E000629 / M

ADJUSTABLE FOOTREST B06 V200/V300/V300DL				Plan No.
				E000629 / M
Scale: 3:10	Drafted	Verified	Confirmed	
Date	28/04/2010	16/12/2024	16/12/2024	
Init.	TM	PM	AS	